

Infrarot Thermometer

Kurzanleitung

Modus

zum Umschalten der Funktionen

1. Körpertemperatur
2. Raumtemperatur
3. Oberflächen-Temperatur

Signalton An/Aus

Taste zum Abrufen der letzten 32 Messungen

Taste 5 Sek. halten löscht den Speicher

Einschalten und Messung durchführen

Messwertanzeige mit farbiger Fieberwarnung

grün = ok

rot = Temperatur erhöht

Messabstand ca. 3-5 cm von der Stirn



TECHNICAL SPECIFICATIONS

1. Normal using condition:

Ambient temperature: 10°C~40°C (50°F~104°F)

Relative humidity: ≤85%

2. Batteries:

DC 3V (2 pcs AA batteries)

3. Unit size: 155 x 100 x 40 mm (L x W x H)

4. Temperature display resolution: 0.1°C (0.1°F)

5. Measuring range:

In body mode: 32.0°C~43.0°C (89.6°F~109.4°F)

Under body mode, there is three color backlit:

Green color backlit:

≤37.3°C (99.1°F), means normal temperature.

Orange color backlit:

37.4°C~37.9°C (99.3°F~100.2°F), means low fever.

Red color backlit:

≥38°C (100.4°F), means high fever.

In surface temp mode: 0°C~60°C (32°F~140°F)

In room mode: 0°C~40°C (32°F~104°F)

6. Precision:

32.0°C~34.9°C (89.6°F~94.8°F) ±0.3°C(±0.6°F)

35.0°C~42.0°C (95°F~107.6°F) ±0.2°C(±0.4°F)

42.1°C~43.0°C (107.8°F~109.4°F) ±0.3°C(±0.6°F)

7. Consumption:

≤300mW

8. Accuracy:

±0.3°C (0.6°F)

9. Measuring distance:

3cm ~ 5cm (1.2in ~ 2in)

10. Automatic power-off:

<30 secs

11. Memory:

32 sets

12. Measuring position:

Forehead skin



Please read the product manual carefully or purchase and use under the guidance of medical staff.
For details on contraindications or precautions.